

Like nobody else



The Power of Personalized Deep Heating

For skin smoothing and cellulite



A new era of personalization


We believe that being different is a good thing, especially when it comes to our bodies.

We all have different body sizes, shapes, weights and goals.

NuEraTight™ has the power to help people celebrate their differences and unleash their bodies' true potential, from head to toe.

NuEraTight is a revolutionary, non-invasive skin smoothing platform, suitable for a wide range of body types and areas. Its state-of-the-art personalized deep heating technology enables to treat a wide range of conditions in one, impactful tailored-made session - delivering natural looking, long-lasting results with 'hot stone massage' feel. All while focusing on simplicity of treatment and providing exceptional ROI.

BODY TYPE: MYSELF

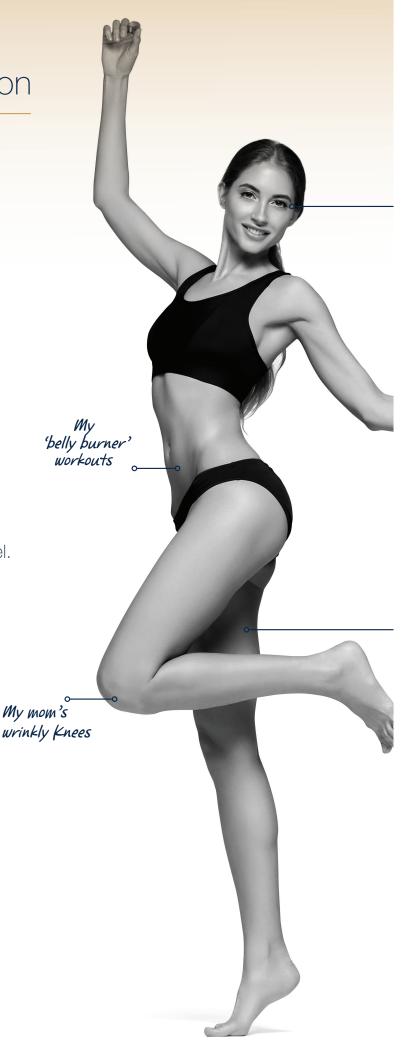

Best-fit for patients. And for you.

Welcome to a new era of fast, effective and versatile treatments for body and face: NuEraTight.

Introduced by Lumenis®, the pioneers in energy-based technologies with 50+ years of innovation, NuEra is a RF device for both superficial and deep heating, to treat a variety of conditions such as temporary reduction in the appearance of cellulite.

What NuEraTight means...

For patients:

- Smooth look
- Treatment comfort
- Satisfying results

For practitioners:

- Simplicity
- Treating a wide range of conditions
- Reliable technology

It's time to help your practice be the best version of itself.

My wrinkles

of joy

My orange peel thighs


Exceptional depth and power: Head-to-toe targeted skin smoothing

NuEraTight is a non-invasive RF solution to improve the appearance of cellulite and skin laxity, with consistent results and without downtime.

NuEraTight emits powerful RF waves at 470kHz with high 250W power, to heat the skin's surface superficially or reach deeper tissue levels. Controlling the depth by using the provided mono-polar or bi-polar hand pieces with a variety of electrode size, you can treat a variety of conditions with different levels of severity and on different areas of the body and face.

NuEraTight optimizes treatment efficacy on the abdomen, flanks, buttocks, inner and outer thighs, upper arms, bra and back fat, as well as around the eyes, mouth, cheeks and neck.

NuEraTight RF emits at a low 470kHz frequency to infiltrate deep down for deeper skin tissue penetration and better thermal effect and results.

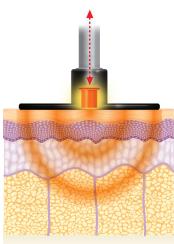




Personalized deep heating: temperature-controlled RF

NuEraTight uses APIC (Automatic Power and Impedance Control), the next generation personalized RF device for both superficial and deep heating, to ensure safety and efficacy.

APIC automatically - and gradually - increases the RF power to the target temperature based on Impedance detection, and then maintains it for the duration of the treatment. It delivers precise heating to the treatment area, with predictable and consistent results.

You can focus on treating your patients with the light and ergonomic handpiece, while letting the system take care of delivering the right temperature and power.

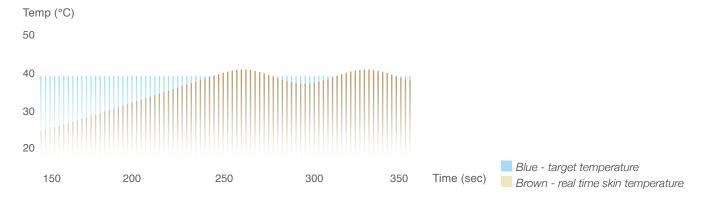

APIC monitors skin tissue temperature and impedance in real time.

It automatically adjusts the RF power so that heat can penetrate deeper and more uniformly - and as needed for each specific treatment

Simply smart precision: How NuEraTight works

NuEraTight is user-friendly: It features an automatic configuration graphic interface for maximum treatment versatility.

Simply select the area of the body or face to be treated, and the system automatically sets the correct parameters, and then displays the right accessory and treatment.

Apply the NuEraTight electrode to the skin's surface with the device set to a specific target temperature. Gently move the electrode in a slow sweeping or circular fashion so that it gradually, uniformly and safely heats up the skin's surface. The treatment is both comfortable to give, and pleasant to receive.

As the skin's temperature rises to the target temperature, the RF output automatically adjusts for precise heating, thanks to the APIC technology. Treatment is fast and efficient, taking around 10-15 minutes to treat facial areas and around 20-25 minutes to treat body areas.





Empower your patient's to love their bodies

Thanks to its remarkable 250W power, NuEraTight uniquely works with one resistive bipolar handpiece and two monopolar handpieces with Coated Capacitive Energy Transfer (CCET) electrodes in a wide variety of capacitor head sizes - from small to very large. The system treats all areas quickly, safely and efficiently, providing natural looking and smooth results.

Patients can look forward to no downtime following an initial course of treatments aimed at achieving their goals. Afterwards, the smoothing effect is easy to maintain with touch-up treatments. This maintenance approach to NuEra treatments keeps patients coming back to your practice and offers a smooth appearance that lasts.

NuEraTight offers a new, 2-step approach to smoothing:



Step 1. GetSmooth

Jump-start with an initial treatment package designed to meet your patient's goals.



Step 2. StaySmooth

Maintain results with a regular treatment touch-up program that keep patients coming back throughout the vear.

All over natural looking results



Before



After



Before



After





After 4 months





After

Courtesy of Concita Muneratti, Luigi rizzi, MD - Italy

Courtesy of Connie Casad, MD - Dallas, TX

Your journey with NuEraTight has just begun.

We want your business to grow with NuEraTight. That's why we have put together a support package that begins with the first conversation and will stay with you as your business begins to see the benefits. We provide a support package which ensures that you have all the training you need and recieve the most up to date clinical materials.







Lumenis Practice Locator

Thousands of patients search for treatment providers every month. Help them find your clinic by registering your practice on the Lumenis Practice Locator. We run campaigns through influencers, social media and viral videos to promote this directory.

Lumenis.com

Consumers have many questions about aesthetic and women's health treatments. Our dedicated patient-facing website will give them the re-assurance and confidence they need to move forward with the treatments.

Team Training

We install the system and train your staff, so that we are able to assess skin types and conditions, creating the perfect treatment plan for each individual patient.

Our expert clinical training means they will handle consultations and treatments with confidence.







Your Most Valuable Resource

Your dedicated representative is an email, text, or phone call away. We want you to feel part of the Lumenis family, so please feel free to contact us any time. We'll always do our best to get back to you within 24 hours.

Brand Your Clinic

As a Lumenis client you will have access to the Lumenis Physician Portal. Here, you can find materials such as brochures and posters. As well as gain access to a wealth of clinical and product information.

Best Practice Marketing

Your individual Lumenis representative will be able to share marketing plans which you can tailor to your own business. These range from 90 day benchmarks for success to loyalty programs and promotions.

Specifications

Emission Frequency	470 kHz ± 10%
Output Energy	250 W Max
Display LCD Touch Screen	15"
Coupling System	Capacitive
Operative Mode	Automatic & Manual
Impedance measurement	Automatic
Termperature Control	Automatic
Temperature Adjustment	37° ÷ 45° ± 1°C
Timer	Max 60 min adj.
Feeding	100-240 VAC, 50-60 Hz
Dimensions	124x57x60 cm (HxWxD)
Weight	30 Kg
Safety Class	Class1 - Type BF
Standard accessories	
Handpieces	3: face and body, bipolar
Return Electrodes	1: grounding pad reusable
Isolated electrodes with temperature sensor	8: ø20, ø30, ø40, ø60, ø70, ø80, ø100mm and Massage electrode ø40
Cosmetics	NuEra RF cream



"NuEraTight is a remarkable supplement to my clinic with unsurpassed advantages combining a well tolerated, pleasant treatment with impressive results."

Clark Schierle, MD, PhD, FACS - Northwestern Specialists in Plastic Surgery Chicago IL, USA

Warnings and Risks: NuEra is unsuitable for anyone who is pregnant, has an infection or illness, or is heat-sensitive. Treatment over cuts, wounds, piercings and tattoos must be avoided and areas with injectable fillers and toxins left for a month before treatment. Patients with a metal implant or implanted device (e.g pacemaker) must consult with a qualified physician first. If deemed suitable, the return pad must be placed far away from the implant. Local, oral or systemic anesthetics cannot be used before or during treatment as patients need to provide regular feedback to ensure their continued comfort. Insulate patients from metal objects, especially any in direct contact (e.g. the bed). A full practitioner consultation is always advisable; for contraindications and risks in full, see the NuEra user manual.

Bios S.r.I.s.u.
Via Guido Rossa
10/12 - 20090
Vimodrone (MI) - Italy
T +39 02 27304275
F +39 02 27304276
info@biosgroup.eu
www.biosgroup.eu

Lumenis (Germany) GmbH Heinrich-Hertz-Str 3 D-63303 Dreieich-Dreieichenhain GERMANY Tel: +49 (0) 6103 8335 0 **EMEA**Dreieich Dreieichenhain,

Dreieich Dreieichennain, Germany T +49 6103 8335 0 F +49 6103 8335 300

Roma, Italy T +39 06 90 75 230 F +39 06 90 75 269 Hertfordshire, UK

T +44 20 8736 4110 F +44 20 8736 4119 JAPAN

Tokyo, Japan T +81 3 6743 8300 F +81 3 6743 8301

AMERICAS San Jose, CA, USA

T +1 408 764 3000 +1 877 586 3647 F +1 408 764 3999 ASIA PACIFIC

Beijing, China T + 86 10 5737 6677 F +86 10 5737 6767

Gurgaon, India T + 91 124 422 07 95

Kowloon, Hong Kong T + 852 217 428 00 F + 852 272 251 51





© LUMENIS LTD 2019





PB-2012101 Rev A

AESTHETIC.LUMENIS.COM